


Product Data Sheet

1.0 Name of the product	Leukoplast® elastic
2.0 Product description	Leukoplast elastic finger dressing Leukoplast elastic dressing length Leukoplast elastic strips
2.1 Description	<p>Leukoplast® Elastic consists of a flesh coloured, woven fabric, extensible in the width, coated directly with a transparent latex free, synthetic rubber-based adhesive mass and perforated by air. A white absorbent wound pad is applied on the coated side of the wound dressing. The adhesive mass of Leukoplast® Elastic is covered totally with white, one side siliconised release paper.</p> <p>Leukoplast® elastic strips consists of a flesh coloured, crosswise elastic Viscose – Polyamide woven which is porous coated on one side with a transparent latex free, synthetic rubberbased adhesive. A white absorbent non-woven wound pad is applied centrally on the coated side of the wound dressings.</p> <p>The adhesive mass of Leukoplast® elastic strips is fully covered with two white, one side siliconized release papers. The lower release paper has got a fold which is overlapped by the upper release paper. Leukoplast® elastic strips is precut to strips, pouched and packed into folding boxes.</p> <p>Leukoplast® elastic dressing length are covered with one release paper, which overlapping the product on both sides, and with a split liner in the middle, wound to rolls (covering outside) and packed into dispenser folding boxes.</p>
2.2 Intended use	<p>Leukoplast® elastic, Leukoplast® elastic dressing length and Leukoplast® elastic strips are intended for short time use as cover for minor wounds like cuts,lacerations, abrasions or as cover of puncture sites after blood withdrawal or vaccination.Especially for joints and awkward body contours e.g. elbows, knees, fingers or wrists.</p>
2.3 Instructions for use	No: Instructions for use not necessary for class I/Is/Ila products.
2.4 Warnings and precautions for use	No warnings and precautions for this device.

2.5 Contraindications	No contraindications are known for this device.
2.6 Transport Precautions	Dry, clean and without exposure to direct sunlight
2.7 Duration of application / Period of use	First aid dressings can be used from one day to several days depending on the status of the wound and the skin condition.

2.9 Composition			
	Designation (incl. no. in drawing)	Description	% of final product
	Backing (1)	70 % viscose / 30 % polyamide	30
	Adhesive (2)	Synthetic rubber-based, latex free	31
	Wound Pad (3)	85% viscose 15% polypropylene / polyethylene	20
Release liner (4)	siliconized paper	19	

2.10 Product range [Packaging /Global Commercial]						
Product	Size	Per folding box		Per shrink	Per shipper	Product code
76455-00003-00	3 x10 cm	50		0	40	76455-03
76455-00004-00	6 CM x 1 m	1		10	120	76455-04
76455-00005-00	6 CM x 1 m	1	10	120	76455-05	
76455-00006-00	6 CM x 1 m	1	10	120	76455-06	
76455-00007-00	8 CM x 1 m	1	10	120	76455-07	
76455-00008-00	8 CM x 1 m	1	10	120	76455-08	
76455-00009-00	Ass.	1	10	120	76455-09	
76455-00010-00	Ass.	1	10	120	76455-10	
76455-00011-00	Ass.	1	10	120	76455-11	
76455-00012-00	28 x 72 mm	20	10	120	76455-12	
76455-00013-00	28 x 72 mm	20	10	120	76455-13	
76455-00014-00	19 x 56 mm	10	10	120	76455-14	
76455-00015-00	Ass.	1	10	120	76455-15	
76455-00016-00	Ass.	1	10	120	76455-16	

76455-00017-00	Ass.	1	10	120	76455-17	
76455-00018-00	Ass.	1	10	120	76455-18	
76455-00019-00	Ass.	1	10	120	76455-19	
76455-00020-00	Ass.NORDIC (20DRS 3SI	1	10	120	76455-20	
76455-00023-00	D = 22 mm	100	0	48	76455-23	
76455-00024-00	19 x 75 mm	100	0	12	76455-24	
76455-00025-00	25 x 32 mm	100		0	48	76455-25
2.12 Storage conditions		Dry, clean and without exposure to direct sunlight				
2.13 Shelf life/Storage time		5 years				
2.14 Sterilization		n.a.				

2.15 Product content list

Substance	Yes	No
PVC in product		X
PVC in packaging		X
Softeners (Phthalates)		X
Natural Latex		X
Natural Latex in Packaging		X
Colophony (CAS 8050-09-7, 8052-10-6, 73138-82-6, 73905-20-1) and derivates		X

History

Version / Date	Page / Item	Description of Change
01/09.2020	All	